# CareStart™ G6PD

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**Meridian Healthcare®**  
medical diagnostics and equipment

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
Test Type	Qualitative point-of-care test
Assay Method	Visual dye colorization method
Interpretation	Normal vs. deficient
Specimen	Whole blood
Specimen Volume	2 µl
Result Time	10 minutes
Package Size	25 or 50 tests/box
Other Specifications	CE Marked, Simple and easy to use, broad range of assay temperature (18-32°C), Room temperature storage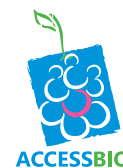

# CareStart™ G6PD RDT

## Test Procedure

**Do not cover hole**


1. Gently hold the pipette below the hole and touch the blood drop~  
**(CAUTION: Do not cover the hole!)**
2. Let the blood flow into the capillary tube.
3. Cover and press the hole to release the blood into the sample well. Add 2 drops of assay buffer (100 µl) immediately after applying blood.
4. Read result in 10 minutes

## Interpretation of the Test Result

### 1. NORMAL

A distinct purple color appears in the reading window within 10 min.


Normal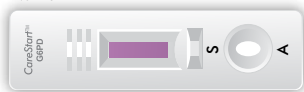

### 2. DEFICIENT

White color or any faint purple should be interpreted as deficient.

Deficient

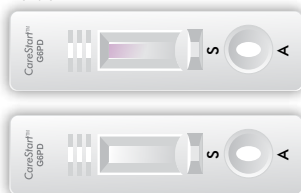### 3. INVALID

No blood migration or incomplete blood migration in window.

Invalid

## Packaging Information

Product	Cat. No.	Package Size	Components
CareStart™ G6PD RDT	RGP-M02582	25 tests/box	Test devices, alcohol pads, lancets, pipettes, assay buffer, instructions for use
	RGP-M05082	50 tests/box	Test devices, alcohol pads, lancets, pipettes, assay buffer, instructions for use