INTERLEUKIN-6

INTERLEUKIN-6 (IL-6) QuickLine

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INTERLEUKIN - 6

Rapid Test

NEW BIOMARKER Rapid Test Lateral Flow

IL-6 is a multifunctional cytokine with a central role in many physiological inflammatory and immunological processes. Due to its major role in initiation as well as resolving inflammation, deregulation of IL-6 is a mainstay of chronic inflammatory and autoimmune diseases.

SEPSIS DIAGNOSIS

due to the relatively slow rise of biological markers such as CRP, which can be in the normal range at the start of an infection, an early diagnosis of sepsis may be not be easy to confirm. However, it is possible to detect a rapid rise in cytokine levels such as IL-6 or IL-8 the clinical relevance of both is comparable) one to two days earlier than CRP; this clearly could improve the outcome for your patient. Even better, the use of IL-6 or IL-8 in combination with CRP allows you to exclude an infection with a very high probability. The savings of unnecessary prophylactic antibiotics in itself leading to a cost reduction for the care of the newborn babe.

INTERLEUKIN-6 (IL-6) QuickLine

The Milenia® QuickLine IL-6 test is a lateral flow immunoassay designed for the semi-quantitative/quantitative evaluation of human Interleukin-6 (IL-6). The sample is pipetted in the sample application port. IL-6 of patient's sample binds to a first monoclonal anti-IL-6 antibody conjugated to gold particles. The IL-6-loaded gold particles diffuse through the membrane and overflow the test line (T). There is a second monoclonal antibody specific for IL-6 coated on the membrane; so gold particles were bound specifically and become visible as a coloured line. Colour intensity is directly proportional to the concentration of IL-6 in the sample and intensifies during incubation time. The surplus of gold particles continues to diffuse over the test unit. The conjugate specific antibodies printed as a line on the membrane (control line, C) captures the rest of gold conjugate and a well visible line develops during the incubation time. The IL-6 concentration correlates directly with colour's intensity of the test line (T).

PERFORMANCE


Rapid Test color immunochromatographic, capillary flow technology.

Sample material: Serum, plasma (EDTA, heparinized, citrated), culture supernatant, amniotic fluid, cerebrospinal fluid.

Sensitivity: 50 pg/ml.

Specificity: Highly specific for human IL-6

Reference values: < 100 pg/mL in healthy blood donors

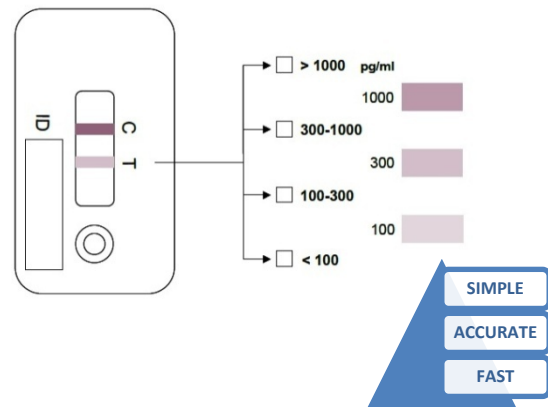Ordering Information:

Interleukin-6 (IL-6)

Product Code: MQL61

Packaging: 20 test

Meridian Healthcare®

Meridian Healthcare srl

68 - Tremestieri Etneo

Catania, IT 95030

Phone: 095 725 6869 (0039)

Fax: 095 725 4454 (0039)